



A · N · I · M · A · L
HOSPITAL SUPPLY

Reorder #1-135 Cage-Duit

Cage-Duit is designed specifically
to fit Shor-Line™ cage doors

STEP 1:



Fit notches of Cage-Duit
through the spokes of the
door with the uncut end
outside the cage

STEP 2:



Affix the 3' insulated hose,
which goes to the blanket
inside the cage, to the end
of the Cage-Duit with
the cut notches

STEP 3:



Affix the blower hose
(outside the cage)
to the uncut end

For best results, attach the Cage-Duit above the top cross bar on the cage door



A · N · I · M · A · L
HOSPITAL SUPPLY

(888) 874.0595

www.animalhospitalssupply.com